



FlopThotics.

Changing Straps

Changing of the straps on your patients
FlopThotics is simple. **EdserLabs®** provides you
with instructions so you can do it yourself. For
Rubber Straps Only!





FlopThotics.


Cambio de tira

1




You will need: The flopthotics, the new straps, scissors, pen/pencil, and expanders.

2

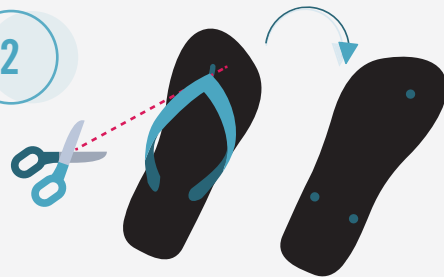

Cut the straps at the three ends, so that it separates from the sole as shown above.

3

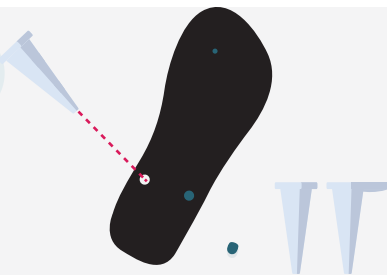

Remove the remaining rubber with the help of the expanders pushing the rubber through.*

4



Repeat the same process with the three holes until all three are empty.

5



Insert the new rubber strap with the help of a pen to exert pressure until it passes through the hole.

6



Repeat the process with the other two ends and you are ready to show off your new flopthotics!

*If you do not have an expander, you can use a pen/pencil or any similar object.

