

EDSER

JERUSALEM - Solid DAFO

The Jerusalem Solid DAFO is Designed to provide a firm and stable base while supporting foot alignment, is a full-contour solid ankle foot orthosis

CLINICAL INDICATED FOR

- Coronal plane instability
- Tight Achilles
- Severe Tip toe walking
- Spastic muscle tone (CP, CVA, or other neurological conditions)
- Flaccid drop foot (when ML stability is required)

FEATURES

The orthosis is designed with variable thickness to provide controlled flexibility and targeted strength. Its low-profile construction ensures a discreet fit that's easy to wear with most footwear.



CONTRAINDICATIONS

- Severe fixed deformity
- Edema/Swelling
- Vascular conditions/Skin risk

INSTRUCTIONS

Casting and/or scanning of the patient is required, and proper positioning of the patient when casting or scanning is fundamental to achieve a correct alignment and the desired outcome. Critical casting and scanning considerations:

- Neutralize the ankle – keep it as close as possible to a 90-degree position
- Correct the rearfoot – Neutralize the Sub-Talar joint
- Correct forefoot varus deformities to achieve a neutral/balanced forefoot position
- Make sure the scan or cast encloses all relevant areas – Plantar surface/Medial and lateral arches/Malleoli/Calf