

EDSER

MALTA - UCBL


The Malta UCBL is designed for stabilization of the heel and midfoot to correct flexible deformities is a versatile UCBL-type orthosis

CLINICAL INDICATED FOR

- Biomechanical control
- Severe overpronation/supination
- Stage III PTTD
- When a normal FFO is not adequate and increased control is required

FEATURES

The deep heelcup stabilizes the heel for better control. The integrated heel post supports proper alignment. Mediolateral reinforcement strengthens the orthotic's sides for correct foot positioning. Customizable lining and padding ensure comfort and a tailored fit

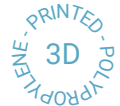

CONTRAINDICATIONS

- Severe fixed deformity of the foot and ankle

INSTRUCTIONS

Casting and/or scanning of the patient is required, and proper positioning of the patient when casting or scanning is fundamental to achieve a correct alignment and the desired outcome. Critical casting and scanning considerations:

- Neutralize the ankle – keep it as close as possible to a 90-degree position
- Correct the rearfoot – Neutralize the Sub-Talar joint
- Correct forefoot varus deformities to achieve a neutral/balanced forefoot position
- Make sure the scan or cast encloses all relevant areas – Plantar surface/Medial and lateral arches/Malleoli/Calf